




TL-3100V Dual Recordable Fall Monitor

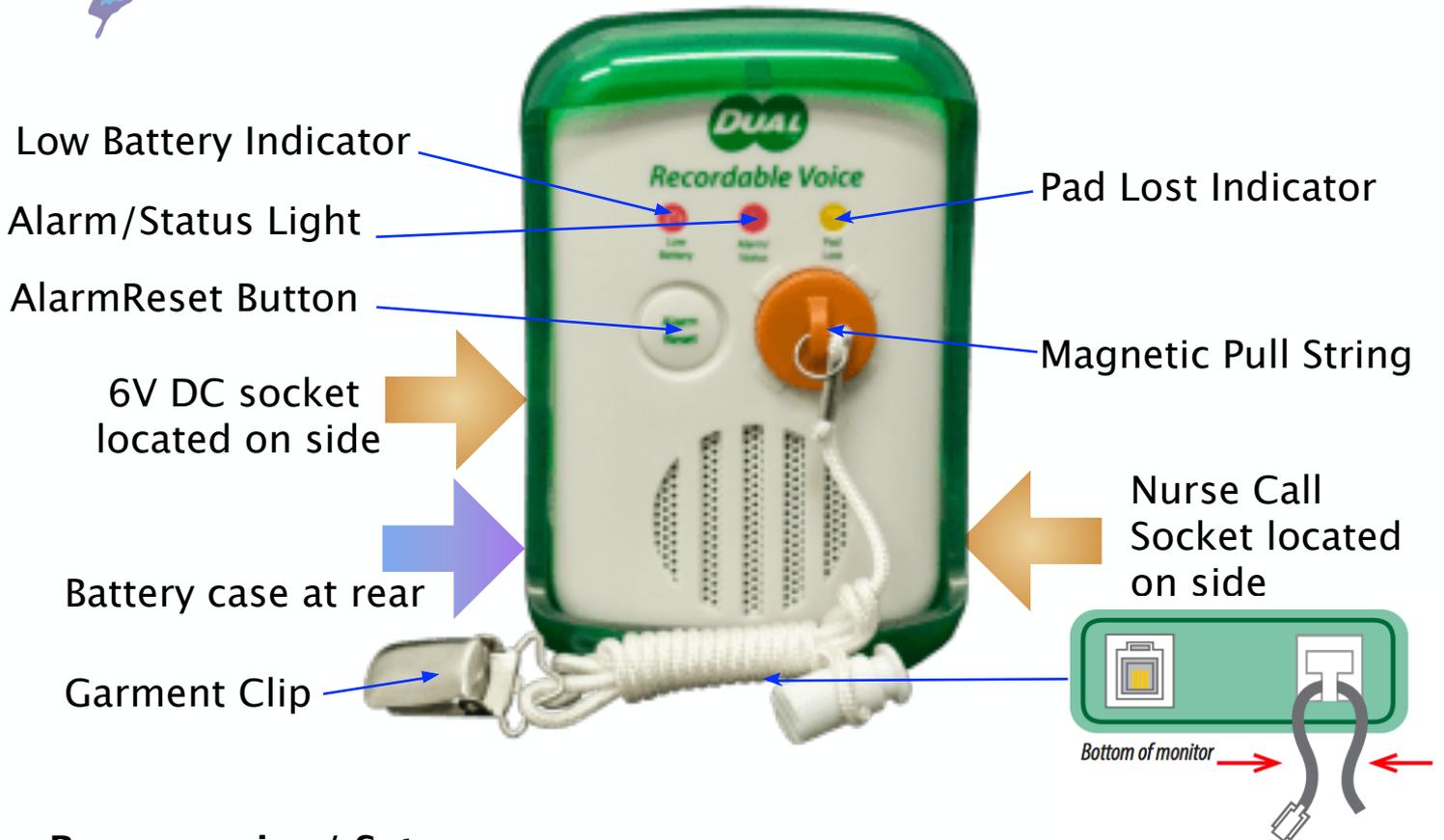

Programming/ Setup

- ◆ Remove protective boot, place 3 x AA batteries in rear (open battery compartment by undoing the screw)
- ◆ Move the "SelectPlay/Record" switch to the record position. Hold the "Play/Record" button to record your message (15 seconds max)
- ◆ After completing recording move the "Select Play/Record" switch to the play position.
- ◆ Adjust the volume wheel to the desired volume
- ◆ **Pull String Operation** – Place orange magnet at front of monitor, attach the monitor to chair/wheelchair then attach garment clip to residents clothing.
- ◆ **Sensor Pad/Floor mat Operation** – Mount monitor to bed or wall, loop the cord from the sensor pad through the strain relief tab on the bottom of the monitor & plug sensor into socket also at the bottom of monitor.
- ◆ Test System

Notes

- ◆ Works as magnet pull string **OR** one bed/chair or mat sensor monitor (not both together)
- ◆ Volume dial, Record/Play switch inside battery case at rear of monitor
- ◆ Recordable voice operation

