


Caregiver Pagers

433-PGD

Install 2 "AA" batteries. Turn pager on – slide power switch to upper position for chime alert or lower position for vibrate alert

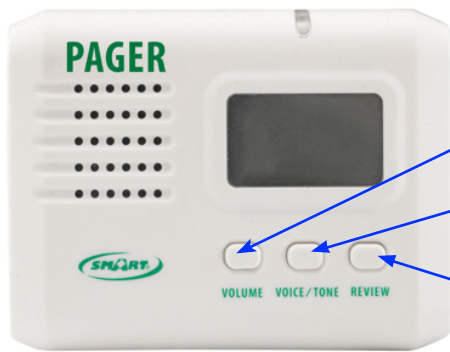

Volume

Voice/Tone

Review


The 433-PGD will work automatically with any 433-CMU Central Monitor Unit within range (up to 70m). The LCD display on the pager will show, by number which device triggered the alert. **No programming required.**



Supplied with case

433-PRB

Install 2 "AA" batteries. Turn pager on – slide power switch to upper position for chime alert or lower position for vibrate alert

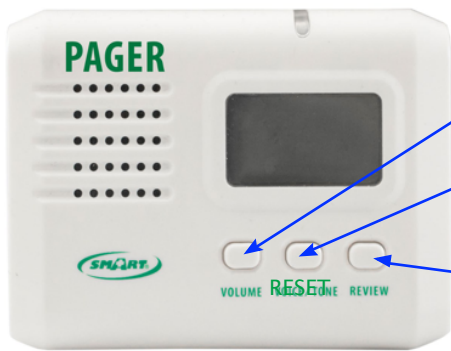

Volume

Reset

Review


The 433-PRB will work with any 433-CMU Central Monitor Unit, when programmed into that unit, within range (up to 70m). The LCD display on the pager will show, by number which device triggered the alert. **Must be programmed into the 433-CMU in the same way as a reset button (refer 433-CMU instructions)**



Supplied with case

TL-2016P

Install 2 "AA" batteries. Turn pager on – slide power switch to upper position for chime alert or lower position for vibrate alert

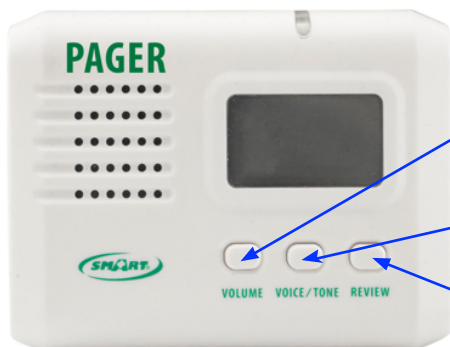

Volume

Voice/Tone

Review

The TL-2016P will work automatically with any the TL-2016 Monitor Unit within range (up to 70m). The LCD display on the pager will show, by number which device triggered the alert. **No programming required.**



Supplied with case