

## Quick Start Instructions

### Step 1

#### TURN MONITOR ON

- Insert 9v battery (not included) into battery compartment. (See diagram).
- The "Status" light will blink every 4 seconds to indicate monitor is operational.

### Step 2

#### PLUG IN PAD

- Plug the pad into the bottom of the monitor.
- Test system. Slide the "On/Off" switch to "On", with no pressure on the pad, monitor will alarm. Please note that the monitor will be shipped with the alarm volume set at 90 decibels (90 decibels is the legally accepted occupational noise allowance).

### Step 3

#### PAD PLACEMENT

- Place pad under shoulders to be alerted when resident sits up in bed.
- Place pad under lower back/buttocks to be alerted when the resident vacates the bed completely.
- Place monitor out of reach of the resident. Suitable mounting locations include: back of headboard, back of wheelchair, wall or under the bed. Make sure that the resident cannot tamper with monitor or reach the "on/off" switch.

### Step 4




#### TURN MONITOR ON

- Turn monitor "On".
- The small power button slide switch located on the side of the unit must be in the "On" position for the monitor to work. It is the responsibility of the caregiver to be sure the monitor is turned on during each use.
- Smart Caregiver Corporation highly recommends the use of the TL-2100S Monitor with "Safety Auto-Reset" so that it is using "always on" technology, thereby removing the need for the "On/Off" switch.

### Step 5

#### TEST SYSTEM

- Apply pressure to the pad to trip the sensor device. The monitor will beep twice.
- The monitor is now active. Alarm will sound when pressure is released from the pad.
- When the alarm is activated slide switch on side of monitor to the "off" position to shut down the alarming mode.
- To reactivate the monitor, slide switch on side of monitor to the "on" position.

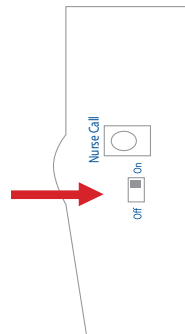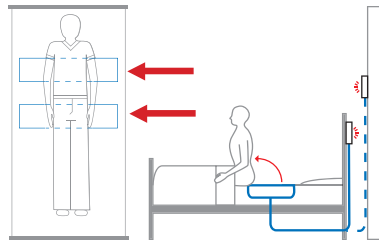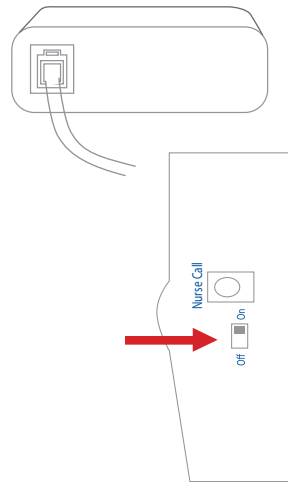

## TL-03 & TL-04 Economy Bed/Chair Monitor

**Caution:** If using this monitor with Change Pad Indicator™ Timed Pressure Pads, monitor will not indicate when pad life has expired. Pads will still function normally. Call (800) 650-3637 for more information.

**Warning:** Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. Test this device before each use. Read the instructions and Legal Disclaimer.

Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.

The Smart Caregiver Corporation Safety Auto-Reset™ Alarm is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good care giving practices including, but not limited to:

- a.) Direct patient supervision
- b.) Adequate training for staff personnel for elopement
- c.) Testing of the system before each use.

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.


**WARNING: The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.**

### Need Set-up Help?

Call Our Friendly Customer Service Reps at (800) 650-3637



1229 N. McDowell Blvd., Petaluma, CA. 94954  
 sales@smartcaregivercorp.com  
 www.smartcaregivercorp.com  
 Toll Free: (800) 650-3637

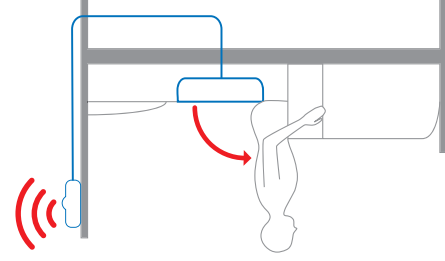

**Installation Instructions**

**Installation Instructions**



**TL-03 & TL-04 Bed/Chair Monitor**



- For use with bed.
  - Lanyard strap.
  - Velcro patch for mounting on wall/headboard.
- TL-03 Contents:**
- For use with chair.
  - Lanyard strap.
  - Clip on back.
- TL-04 Contents**

Warranty.

This Bed and Chair Fall Alarm Monitor is for use with bed or chair fall alarm pressure pads. When the resident attempts to vacate the bed the alarm will sound. The removal of weight from the fall alarm pressure pad causes this fall alarm to sound so that the caregiver may attend to the resident. Caregiver must deactivate the fall alarm by turning the unit off and then reset by turning the fall alarm unit on again. This unit has a One Year

**TL-03 & TL-04 Economy Bed/Chair Monitor**



1229 N. McDowell Blvd., Petaluma, CA. 94954  
 sales@smartcaregivercorp.com  
 www.smartcaregivercorp.com  
 Toll Free: (800) 650-3637