

# TL-2100B Fall Monitor



Low Battery Indicator

Reset Button


Connect sensor pad at top of monitor

Status Indicator

9V DC socket located on side

Nurse Call Socket located on side

Battery case at rear

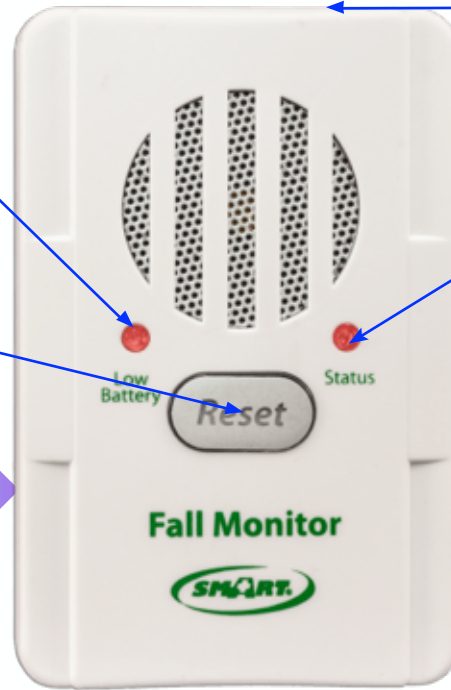

## Programming/ Setup

- ◆ Place 9V battery in rear battery compartment (remove screw to access)
- ◆ Establish the fall monitor settings with the switch inside the battery compartment (silent, low or high) **Note: Silent should only be selected when the TL-2100B is to be used with a facilities existing Nurse Call Light System.** Close battery compartment.
- ◆ Connect one corded pressure pad (socket located at top of monitor – run cable through strain relief recess)
- ◆ Testing – apply pressure to sensor pad to activate. The monitor will beep twice to indicate the sensor pad and monitor are in use. When pressure is removed from pad, alert tone will sound. Press the reset button on front of monitor to silence / reset.

## Notes

- ◆ For use with one corded bed pad or chair pad.

