


# TL-2100CP Change Pad Indicator Fall Monitor

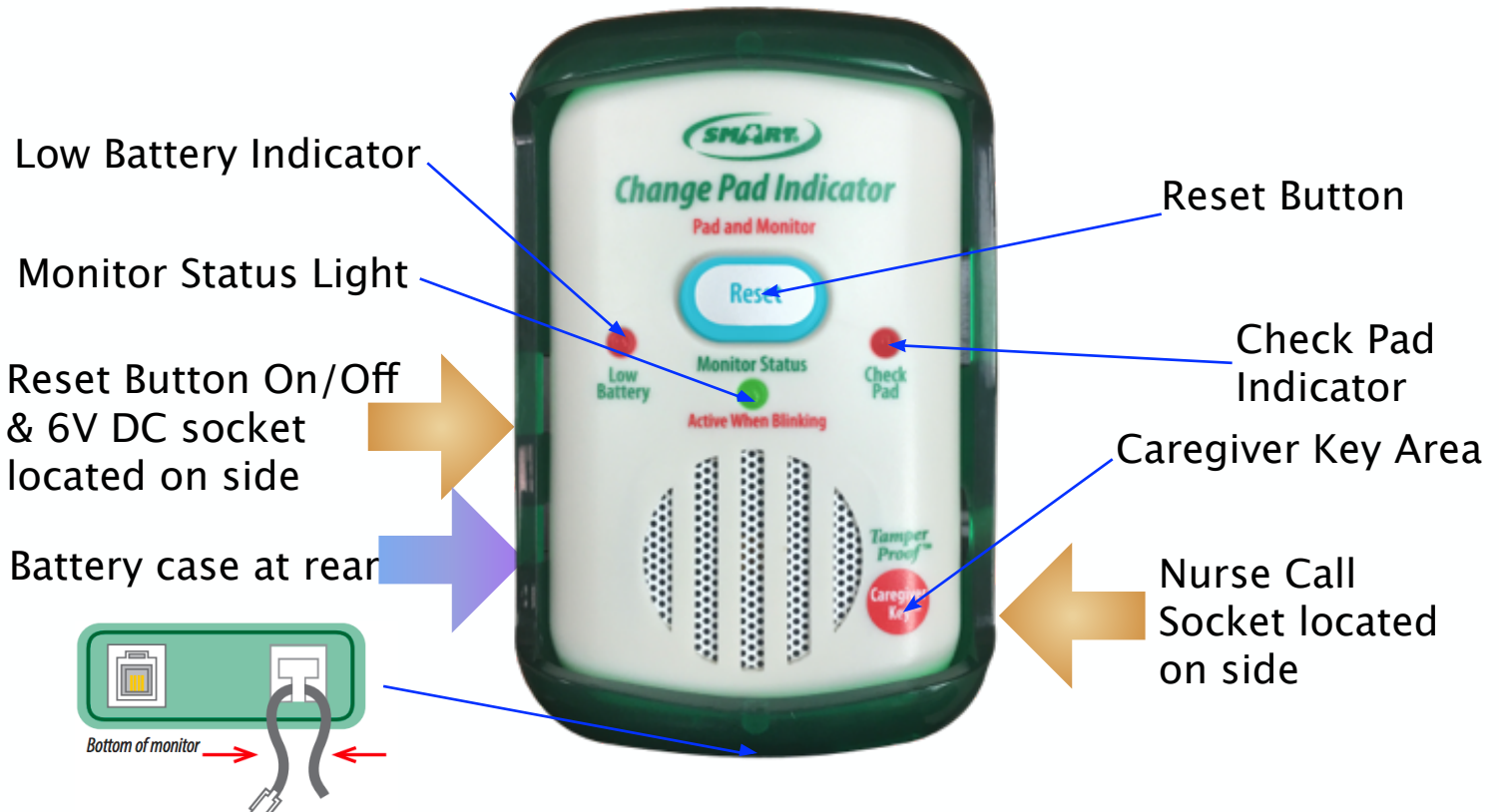

## Programming/ Setup

- ◆ Remove protective boot, remove screw from battery cover, place 3x AA batteries in rear & adjust volume, tone and delay. Close battery compartment and tighten the screw. Replace boot.
- ◆ **Connect** one corded "Timed" sensor pad in socket located on bottom of monitor – run cable through strain relief recess.
- ◆ Testing – apply pressure to the sensor pad to activate. Monitor will beep twice, Status light will blink while pressure remains on pad. When pressure is removed the alert tone will sound. Press the reset button to silence/reset.

## Notes

- ◆ For use with one corded timed bed pad or chair pad. Will work with sensor mat but timed change pad feature will not work.
- ◆ Volume dial, tone settings and delay switch inside battery case at rear of monitor
- ◆ Reset switch on side of monitor – "Reset Switch On" means the reset button on front of monitor is active. "Reset Switch Off" means Caregiver Key point at front of monitor is active, and reset button on front is inactive.
- ◆ The TL-2100CP has an internal clock and measure time the pad is in use (switched on with pressure applies) 10 days before sensor pad life expires check pad light illuminates and monitor chirps.
- ◆ Designed to be used with the timed change pad sensor pads. Will work with ordinary Smart Caregiver corded pads but timed change pad function won't work